Eyewash bottles BARIKOS

First Aid Measure!




Eyewash bottles are used as a first aid measure to rinse the eyes in order to remove penetrated substances, e.g. dust or chemicals. They should always be at hand and ready for use at hazardous workplaces.

The types BARIKOS and BARIKOS K are designed to be filled with clear water.

The BARIKOS KS and Mini BARIKOS KS models are maintenance-free versions of eye-wash bottles. Containing gamma-radiation-treated water, they can be kept ready for use for at least two years in the sealed state. Both models are ideal for keeping in first aid boxes and sets. BARIKOS K and KS eyewash bottles comply wit the standard DIN 12930.

8-450 EN 05/2005



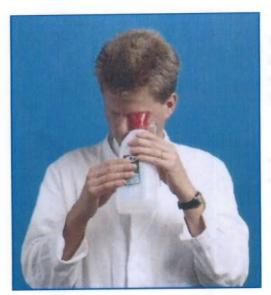

Using an eyewash bottle in the upright position we recommend the type BARIKOS (vol. 620 ml). The user only is obliged to turn his head a little bit downwards coming into good contact to the spray head of the bottle.

BARIKOS: P/N: 954000

The Mini BARIKOS eyewash bottle (vol. 175 ml) is available for persons, being frequently present in different working areas. It can be easily carried along in your workdress pocket.

Mini BARIKOS: P/N: 954003

The BARIKOS K and KS eyewash bottles (vol. 620 ml) can be used immediately even when the injured person is lying down

As soon as the bottle is tilted downwards, a patented sliding valve opens and the washing fluid reaches the spray head unhindered.

BARIKOS K: P/N: 954002

A protection cover tightens the spray head of the eye wash bottles BARIKOS KS in a hygienically way and without any leaking. The unbroken seal guarantees proper function. Shelf life of the eye wash bottle always is printed on a sticker on front- and backside of the bottle.

BARIKOS KS: P/N: 954004 Mini BARIKOS KS: P/N: 954005





Keeping eyew ash bottles everytime in the "ready for use" stage, we recommend the following possibilities:

- fixation support, made of plastic coated steel wire, designed for one eye wash bottle.

 P/N 154808
- plastic cabinet for wall assembly for one eye wash bottle; the transparent cover is marked with the First-Aid-Symbol.
 P/N 154812
- plastic cabinet for wall assembly for two eye wash bottles;
 the transparent cover is marked with the First-Aid-Symbol.
 P/N 154809

BartelsRieger Atemschutztechnik GmbH & Co. KG Richard-Byrd-Straße 23 · D-50829 Köln (Ossendorf) Telefon +49 (0)221 59777-0 · Fax +49 (0)221 59777-159 www.bartels-rieger.de · e-mail: barikos@bartels-rieger.de



Data Sheet: Eye wash bottle Mini BARIKOS KS

BR-Article-No.:

954005

Trade name:

Mini BARIKOS KS

Product-description:

Eye wash bottle according EN 15154-4:2009-07

with filling free of germs

Kind of product:

First-aid equipment for eye rinsing

Kind of use:

Eye rinsing in standing up-, sitting- and lying position as first aid self

help measure to wash out foreign matter or substances

Description:

Transparent plastic bottle with printed on pictogram, with red spray

head and white cap, internal ascending tube with slide valve,

protection seal

Contents:

Drinking water (ph-value: 7 - neutral), 175 ml,

treated by gamma rays, free of germs

Materials:

Bottle, protection cap:

Polyethylene

Polyethylene

Tube:

Spray head, angle piece,

slide valve body:

Polystyrene

Slide valve ring:

Polymethylene oxide

Washer:

Nitrile rubber

Measurements:

approx. 80 x 30 x 180 mm

Weight:

approx. 250 grams

Marking:

Mini BARIKOS KS – EN 15154-4 - 175 ml – Charge "xx" (Charge-No.) – "yyyy/mm" (end of "good for use"-time) – 954005 (Article-No.)

Handling/Storage:

At room temperature!

Protected from cold, heat and direct solar radiation!

Warning advice:

Eye rinsing does not replace an ophthalmological examination

DS 8-502 16.11.2009

